

VACUMATIC®

Vacuum Technology

With Vacumatic vacuum technology, you can guarantee hygienic waste disposal, while facilitating a pleasant living and working environment.



The Vacumatic Technology

Vacumatic technology removes the air from waste bags and then hermetically seals them removing the problem of exposed bacteria, viruses and unpleasant odours. The Vacumatic system provides a hygienic, easy to operate solution to bad odour in health care facilities.

The patented bags that are used in the Vacumatic system have a strong wall structure which prevents leakage of air or liquid after sealing. They are available in clear or yellow & black 'tiger' bags.

The Vacumatic system aids collection, storage and transportation of incontinence and other clinical waste in a safe, easy to operate and economical way.

If your facility relies on incontinence products, Vacumatic allows you to dispose of waste quickly, easily and hygienically.

The Vacumatic system comprises of multiple collection trolleys and a central vacuum unit which once each special bag is full, it will be compressed and heat sealed to prevent spillage or odour escape.



Benefits

- Hands free operation reducing the risk of cross-contamination
- Suitable for use in combination with all types of incontinence products
- Unit is fitted with bacteria and odour filters for the protection of staff and the environment
- The strong Vacumatic bags negate the risk of splitting during movement to outside garbage areas and reduces odour whilst waiting for collection
- Plugs into standard electric wall socket



info@ddcdolphin.com



www.ddcdolphin.com

VACUMATIC®

Vacuum Technology

VACUUM SEALER

By placing the vacuum bag into the machine the air is extracted from the bag, and then hermetically sealed using a heating element.

VACUUM BAG

The sturdy vacuum bags which are 0.004-inch thick are odourless and airtight once sealed

AIR FILTERS

The air drawn from the vacuum bag is filtered through a bacterial filter, then through an odour filter.

EASY TO LOCATE

The Vacumatic is mobile and easy to move.

ACTIVATION SENSOR

The Vacumatic is opened by simply moving the foot under the light sensor. This allows it to be operated hands-free. The process is completely safe and automatic.

DDC Dolphin Order Code	Description	Qty
VAC0001	Vacumatic	1



info@ddcdolphin.com



www.ddcdolphin.com

Vacumatic Trolley

The Vacumatic Trolley has been specifically designed as part of the Vacumatic waste disposal system.

The Vacumatic Trolley uses a unique rubber clamping system operated by a foot pedal which opens the clamp spring top and closes automatically when the foot is removed. This means that the waste bag is never left open, preventing any bad odours from escaping, keeping your facility completely safe, efficient and odour free.

DDC Dolphin Order Code	Description	Qty
VAC0007	Vacumatic Trolley	1



Vacumatic Bags

Vacumatic bags are strong (0.004" thick material) and have a patented structure to maintain strength. They will not allow the escape of liquids or foul odour once sealed.

DDC Dolphin Order Code	Description	Qty
VAC0009	Vacumatic Trolley Bags - Clear	100
VAC0010	Vacumatic Trolley Bags - Tiger	100



VACUMATIC®

Vacuum Technology

The Vacumatic is available to purchase on a versatile and affordable payment plan.

Our 360° Vacumatic package allows you to spread the cost - not infection!

For a manageable monthly fee, 360° Vacumatic **prevents infection, protects health** and helps you **preserve your budget** too.



360° Vacumatic package includes:



PLUS: Full ownership of the machine after 5 years.

*Warranty policy applies

This document has been carefully formulated to be a true representation of the product, however the manufacturer accepts no responsibility for any changes, or for any consequence arising from this document.



info@ddcdolphin.com



www.ddcdolphin.com